



Give More Care for LESS COST ... The LEEDer way

INSTRUCTIONS VCO

VCO – Vertebral Column Orthosis

Order Code = LG-VCO (Small, Medium, Large)

- **Vertebral Column Orthosis Helps immobilize spine stabilizing weak or injured structures and deformities; Provides stabilization of the lumbo-sacral spine; Facilitates healing of injuries or after surgical procedures; Reduces Pain**
- Body Jacket design compresses Upper Torso and Abdomen restricting mobility, unloading discs, vertebrae and other spinal structures
- Flex-E-CoreTechnology design allows “hands-on” fitting; unlimited adjustment.
- No Heat or Special Tools Required
- Easy don / doff with patient Quick release
- Worn Under or Over Clothing
- Two Models: Padded and Thin
- Multiple Sizing Selection
- Right and Left Hand “Gaiting” / transfer straps included; wheelchair security

ORDER CODE = LG-VCO NEW DESIGN

1. Unpack VCO and disconnect straps.
2. Measure curvature of patient and adjust VCO pads to accommodate.
3. Start with Posterior Pad. Use a goniometer to transfer the desired angles. Pads may be adjusted by hand. Place pads over a rounded surface (bed rails, etc.) to bend.
4. To reduce the potential for skin break down have the patient wear an under-garment. With the pads adjusted to fit, hold the Posterior pad in place and adjust the side straps on the side **opposite** the quick release connectors so that the Anterior pad is centered on the abdomen.
5. When the pad is centered, engage the quick release connectors and tighten the straps.
6. Tighten the straps in sequence to apply pressure as desired and to keep the pads properly positioned.
7. After the straps are adjusted use the quick release connectors to don / doff.
8. Ambulation Straps: Apply the ambulation straps to the back “d” rings as necessary.
9. Laundering: carefully remove the fabric cover from the Flex-E-Core frame. Bend the frame as necessary to slip the fabric cover on or off. Launder with mild detergent, cool water and cool dry.

Toll Free Customer Service 866-814-0192

www.LEEDerGroup.com

Toll Free FAX 866-818-0373

LEEDerGroup@LEEDerGroup.com

8508 NW 66th St Miami FL 33166